VICEGRIP **Immobiliseerder**

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PO Box 13897
Northmead
1511
Justin.frewen@absamail.co.za

VICEGRIP IMMOBILISER RANGE:

			
	★ ★ ★	★ ★ ★ ★	★ ★ ★ ★ ★
Product Code	041071	041169	041134
Immobilising Unit	✓	✓	✓
Blamps (Black & Red)	✓		✓
Charging Cable	✓	✓	✓
AC/DC Adaptor	✓		✓
Probe		✓	✓
Lifetime Guarantee*	✓	✓	✓

*See Terms and conditions on last page of this document

BEFORE YOU START

1. Introduction:

The *VICEGRIP* Immobiliser is a safe electronic apparatus designed to immobilize the voluntary muscles of livestock. It is safe and harmless to the animal and handler when used correctly.

There are many different views on the best method of immobilization (cable and clamp or rectal probe) and because animal's behavior can vary from one to the next Antrovet promotes both with one condition:

DO NOT use the rectal probe for procedures on the front of the animal

Due to the distance from the rectum to the head, the effect of immobilization with the rectal probe is not as effective as with the cable and clamp set. The cable and clamp set uses two electrodes on the front and back of the animal in contrast to the rectal probe where both electrodes are in the rectum of the animal.

2. Charging of Battery:

- The *VICEGRIP* Immobiliser is fitted with a nickel cadmium re-chargeable battery and can be charged from any 12 Volt cigarette lighter socket with the charging cable provided. It can also be charged from a 220 Volt outlet with the AC/DC adapter (included in the 3 & 5 star models)
- The built-in battery charger switches to trickle mode when battery is fully charged and can therefore not be overcharged.
- The battery will reach full charge between 10 – 16 hours.
- **IMPORTANT: Please charge battery for 16 Hours before first time use!**
- Spare batteries as well as 220 Volt chargers are available from Antrovet.

BEFORE YOU START continued

3. Suitability for your animal and procedure?

- The *VICEGRIP* Immobiliser is suitable for a wide range of normally labour intensive and inherent dangerous procedures e.g.:
 - Castration
 - Dehorning
 - Branding (hot and freeze)
 - Ear notching
 - Tattooing
 - Nose ring application
 - Hoof Care
 - Sheath Washing
 - Many more...
- This device only immobilizes the animal – it does not have any anesthetic effect!
- This device is **NOT** suitable for procedures of surgical nature or procedures exceeding 5 minutes.
- Do **NOT** use *VICEGRIP* Immobiliser on:
 - Animals suffering from any systemic disease
 - Horses
 - Humans

4. Reading of instructions and testing of your unit


- The instruction are divided into 2 sections please read the relevant instructions thoroughly **BEFORE** you attempt the procedure
 - Instructions: cable & clamp set
 - Instructions: use with rectal probe

INSTRUCTIONS: CABLE & CLAMP SET

1. Ensure animal is in an area where you have control over it (holding pen preferably with suitable neck clamp. Avoid slippery ground surfaces).
2. Secure *VICEGRIP* Immobiliser on top railing of holding pen.
3. Run cable to back of the animal in such a way that cable cannot be hooked or damaged by animal.
4. Test the Unit:

- Connect cable and clamp set to *VICEGRIP* Immobiliser
- Switch on power – you will hear a whistling sound from alarm
- Clamp red and black clamps together
- Alarm should stop – this indicates that the signal is going through and unit is ready for use.
- Should the alarm not stop – check for a loose or faulty cable

5. Set power level to ZERO. Wet both electrodes with water.

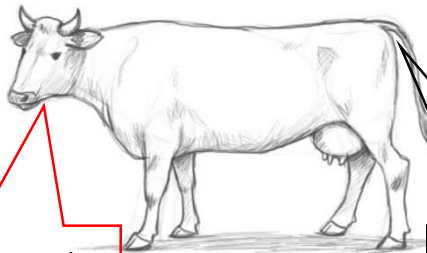Connect red clamp to corner of the mouth – ensure contact with inner cheek

Connect black clamp to the **opposite** side, under the tail, on the tail root

7. Keep an eye on the animal and increase power setting until animal arches its back. The animal may:
 - Roll its eyes
 - Breathe heavily
 - Bellow
 - Salivate excessively
8. Monitor animals **breathing!** If breathing stops for longer than 15 seconds decrease the output level. Be careful not to decrease it below a level where all immobilization is lost while procedure is carried out.
9. Monitor time! The *VICEGRIP* Immobiliser should **NOT** be used on an animal for longer than **5 minutes!**

✓ Tips for Safe Settings:

- Use the lowest settings possible to avoid build up of lactic acid
- A mature bull has a fully developed central nervous system and therefore will need a lower setting than a calf with developing nervous system.
- Handlers normally learn from experience the best settings for his / her animals. If a setting of more than 14 is needed to sustain immobilization or the alarm whistles continuously, there might be a loose connection or insufficient contact between clamp and skin of animal – try to wet the clamps.

INSTRUCTIONS: USE WITH RECTAL PROBE

1. Ensure animal is in an area where you have control over it (holding pen preferably with suitable neck clamp. Avoid slippery ground surfaces).
2. Secure *VICEGRIP* Immobiliser on top railing of holding pen.
3. Run cable to back of the animal in such a way that cable cannot be hooked or damaged by animal.
4. Test the Unit:
 - Connect rectal probe to *VICEGRIP* Immobiliser
 - Switch on power – you will hear a whistling sound from alarm
 - Submerge metal part of probe in a bucket of water (ensure to keep your hand piece out of the water)
 - Alarm should stop – this indicates that the signal is going through and unit is ready for use.
 - Should the alarm not stop – check for a loose or faulty cable
5. Insert probe into rectum. Do not insert probe past the hand piece collar.
6. Keep an eye on the animal and increase power setting until animal arches its back. The animal may:
 - Roll its eyes
 - Breathe heavily
 - Bellow
 - Salivate excessively
7. Monitor animals **breathing!** If breathing stops for longer than 15 seconds decrease the output level. Be careful not to decrease it below a level where all immobilization is lost while procedure is carried out.
8. Monitor time! The *VICEGRIP* Immobiliser should **NOT** be used on an animal for longer than **5 minutes!**

✓ Tips for Safe Settings:

- Use the lowest settings possible to avoid build up of lactic acid.
- A mature bull has a fully developed central nervous system and therefore will need a lower setting than a calf with developing nervous system.
- Handlers normally learn from experience the best settings for his / her animals. If a setting of more than 14 is needed to sustain immobilization or the alarm whistles continuously, there might be a loose connection.
- The use of the rectal probe tends to be less traumatic for animals. It is especially handy for use in animals that resist attaching clamp to lip area and where branding on hind leg is done.
- Clean probe and clamps before using on another animal to prevent transmission of disease.

TROUBLE SHOOTING

PROBLEM	POSSIBLE SOLUTION
Animal stops breathing for longer than 15 Seconds	Output level set too high. Lower the power output setting.
Alarm does not stop whistling when black and red electrodes are pressed together for unit test	<ol style="list-style-type: none">1. Cables are not properly attached to unit – check and attach2. Cable is faulty – replace cable3. Battery could be flat – charge battery
Alarm sounds suddenly during procedure	Check clamps on animal. Re-clamp if they are loose
Battery indicator light is on	Battery needs to be charged
Battery only holds charge for a few animals	Battery needs to be replaced

Spare cables, batteries and other parts are available from Antrovvet.
220 V chargers are also available

VICEGRIP TESTING PROCEDURE

1. Switch device on. The “IMMOBILISER ON” light should be on and the “BATTERY LOW” light should be off. If th“BATTERY LOW” light is on it means battery is too low to work effectively. Charge device. If light still stays on after charge the battery needs replacement.
2. With device switched on, turn the dial from 2 to 24 with no cables connected the device should sound an alarm from 3 through 24. Create a short between the 2 pins and alarm should cease (Photo 1).
3. Connect the short cable to device. With 2 ends open alarm should sound and with 2 ends touching alarm should cease (Photo 2).
4. Now connect clamp cables to short cable. With 2 clamps not touching alarm should sound and with 2 clamps touching alarm should cease (Photo 3). If this is not the case, cables might be faulty .
5. The same test can be done on the rectal probe. Connect the probe cable to device. Make a “short” between the front and back surfaces of the probe. With 2 ends touching alarm should not sound (Photo 4).

GUARANTEE

- There is a lifetime guarantee on the electronics of the *VICEGRIP* Immobiliser.
- Exclusions:
 - Batteries, cables, chargers, printing, covers and containers.
 - In the event that electronic components of the product have become obsolete or unavailable due to advances in technology.
 - Indications that the product was exposed to liquid or other foreign substances or damage that is not considered to be as a result of component failure or normal wear and tear.
 - Unauthorised alterations or attempted repairs by persons other than our service staff

We are proud of our products and service!
Please contact us for any product related information or questions